## VIII. Proper Care And Maintenance

### STORING THE DURA-NEB 3000

If you plan on storing the DURA-NEB 3000 for an extended period of time, recharge the internal battery overnight once every 2 months, unplug the compressor, then return it to storage. Under normal circumstances, as the battery nears the end of its life, it will require recharging more frequently. Contact your DURA-NEB 3000 dealer or PARI Patient Connection to purchase a replacement battery (part# 20F1000) when you notice diminished battery capacity. Changing the battery is strongly recommended when the battery is not operating properly. If the new battery does not correct the problem, then contact PARI Patient Connection for service, (see section IX).

Note: Excluding battery, there are no user serviceable parts inside the DURA-NEB 3000 compressor case. Service by a non-authorized DURA-NEB 3000 service center voids warranty.

### CLEANING THE DURA-NEB 3000 HOUSING

Use a damp cloth to wipe the exterior of the DURA-NEB 3000 housing. Do not use any commercial cleaners on the unit.

### MAINTENANCE

Never operate the DURA-NEB 3000 if it is damaged or improperly or incompletely assembled. Use only original PARI parts and attachments authorized by PARI for the DURA-NEB 3000. Use of other parts and attachments may damage the unit or cause improper treatment. Do not open or disassemble the compressor unit (except for the Battery Access Door) at any time. Repairs, service and maintenance should be carried out only by an authorized PARI service center.


### CLEANING THE TUBING

If there is any moisture or condensation in the tubing, let the compressor run with only the tubing attached for 10 minutes. The warm air from the compressor will dry the moisture or condensation. Alternatively, you can dry the tubing by removing it from the compressor and hanging it up with the "ends down" [E-1] to allow any moisture or condensation to drain out and evaporate. Use a damp cloth to wipe the exterior of the tubing.

### FILTER REPLACEMENT

Under normal conditions, the filter on the front panel of the DURA-NEB 3000 should be replaced after 500 hours of use, or sooner if it appears soiled (grey), clogged or wet. Check the condition of the filter after every 10-15 treatments. Do not attempt to wash and re-use filters. Do not use substitute filter materials as compressor damage can result. Any attempt to use alternative filter materials will void compressor warranty and cause damage to the compressor. When replacing the filter, make sure that the front panel ON/OFF switch on the DURA-NEB 3000 is in the "OFF" position. Turn the round plastic filter cover counterclockwise (left) and remove filter (use a coin to help turn the cover if tight). The entire filter assembly was designed as a remove and replace item [E-2]. That means removing the filter, discarding it, and replacing it with a new filter assembly (part# 41F64).

Note: Do not attempt to disassemble filter assembly. Do not attempt to wash and re-use filter material. Do not attempt to replace the filter material only.

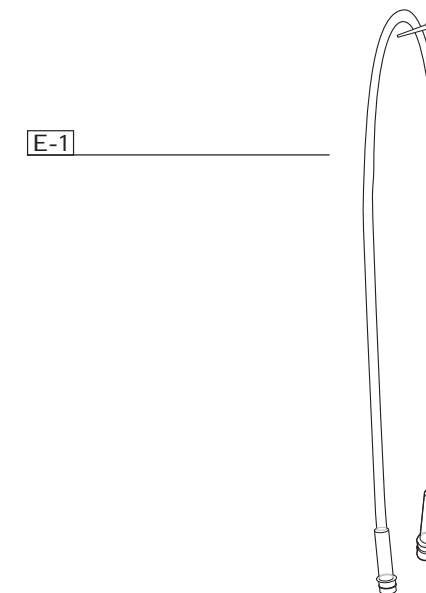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