III. Specifications

COMPRESSOR:

Weight (with battery):	5.0 lbs. (2.3 Kg)
Weight (without battery):	2.4 lbs. (1.1 Kg)
Dimensions:	3 7/16" x 6 1/8" x 6 7/8" (8.7 x 15.6 x 17.5 cm)
Ambient Temperature Range:	50° F - 104° F, (10° C - 40° C)
Maximum Pressure:	25 ± 5 psi

AC/DC CHARGER/ADAPTER:

Input Voltage Range:	90V AC to 265V AC
Input Frequency:	47-63Hz
DC Output Voltage:	15V DC
Rated Output Current:	2.0A
Wattage:	30W
Insulation:	Grounded

INTERNAL BATTERY:

No. of Treatments Per Charge:	Up to 18 treatments when nebulizing 2.5 ml of medication using a PARI nebulizer
Charge Time:	8 Hrs. (initially and when depleted)

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IV. Getting started

1. Check the DURA-NEB 3000 packing box and make sure all components listed below are included.

DESCRIPTION:

	PART NO.
DURA-NEB 3000 Portable Aerosol System (With Battery)	20F30-LC+
DURA-NEB 3000 Portable Aerosol System (Without Battery)	20F31-LC+
Deluxe Carrying Case	20F2999
Multi-Voltage AC Charger/Adapter	20F6000
DC Charger/Adapter (cigarette lighter or vehicle outlet use)	20F2036
Air Filter (in compressor)	41F64
DURA-NEB 3000 Instructions For Use	20D0005
PARI Nebulizer Set	22F81

OPTIONAL ACCESSORY:

Internal Battery	20F1000
------------------------	---------

If you did not receive all of the components listed, please contact the authorized dealer where you purchased your DURA-NEB 3000.

2. Fill out warranty card completely and mail.
3. It is recommended that the customer purchase a back-up nebulizer in case of lost or damaged parts.
4. Contact PARI Patient Connection at 1.800.FAST.NEB (327.8632) and Press 2 if you have any questions about getting started.

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