## VI. Set-Up

## CHOOSE THE APPROPRIATE POWER SOURCE

The DURA-NEB 3000 can be used with the following power sources:

- Rechargeable Internal Battery (optional)
- Multi-Voltage AC Charger/Adapter For Worldwide Use
- 12V DC Cigarette Lighter or Vehicle Outlet

**A TO USE THE DURA-NEB 3000 FROM THE WALL OUTLET (115V OR 230V)**

1. Connect the plug of the AC Charger/Adapter into the wall outlet **A-1**.
2. Plug the small round end of the AC Charger/Adapter into the socket on the front panel of the DURA-NEB 3000 marked "Power Input" **A-2**.
3. Read and follow the Instructions For Use included with your PARI nebulizer package.


Note: The Multi-voltage Charger/Adapter comes equipped with an American Pattern Plug. You may need an Adapter Plug Set **A-3**, (sample plug shown) for most European Countries (Part No. 44F2207). The Charger/Adapter automatically converts the incoming voltage of the country visited. Manual switching is not required.

**B TO USE THE DURA-NEB 3000 FROM A 12V DC CIGARETTE LIGHTER OR VEHICLE OUTLET**

1. Plug the cigarette lighter end of the DC Charger/Adapter into the 12V DC cigarette lighter or vehicle outlet in the car, boat or motorhome **B-1**.
2. Plug the small round end of the DC Charger/Adapter into the socket on the front panel of the DURA-NEB 3000 marked "Power Input" **B-2**.
3. Read and follow the Instructions For Use included with your PARI nebulizer package.

**C TO USE THE DURA-NEB 3000 FROM THE INTERNAL BATTERY**

1. You must fully charge the internal battery before using for the first time. To charge the internal battery, follow the steps in Section VII.
2. As you take your treatment, please pay attention to the Battery Capacity Light so that the battery maintains a sufficient charge. The Battery Capacity Light will be green when the battery is fully charged and it will turn to yellow as the battery is depleted. When the Battery Capacity Light is red, the battery will need to be recharged (see Section VII). The battery will last longer if you recharge the unit as often as possible. Operating compressor when capacity light is red may damage battery. AVOID DRAINING THE BATTERY COMPLETELY, AS THIS WILL LOWER THE LIFE EXPECTANCY OF THE BATTERY.
3. Read and follow the Instructions For Use included with your PARI nebulizer package.

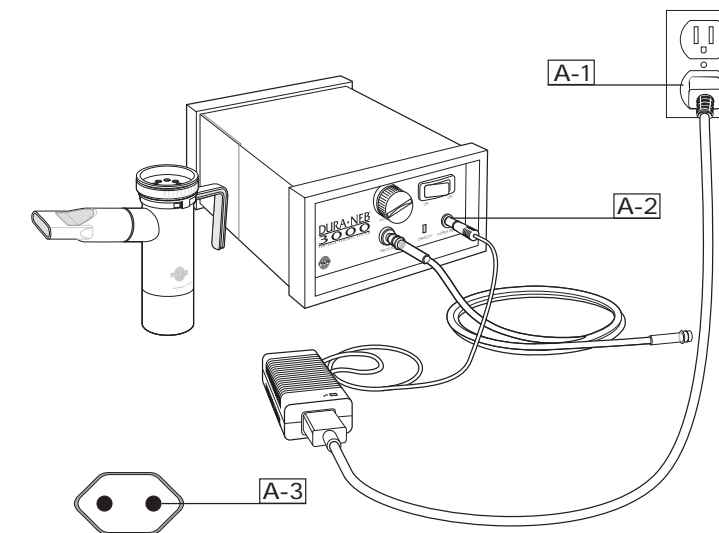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