VII. Internal Battery Care (for 20F30-LC+ only)

Your battery is a gel, suspended cell battery, which does not require you to drain it completely before recharging. The battery will last the longest if you can recharge the unit after every treatment. The more you recharge the battery, the longer it will last. When the battery is new, the DURA-NEB 3000 will deliver up to 18 treatments when nebulizing 2.5 ml of medication using a PARI nebulizer. As time goes by, the battery capacity will diminish. The life of the battery is affected by how often you recharge it and how long you wait before recharging. The optimum care for the battery is to recharge it as often as possible.

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TO RECHARGE THE INTERNAL BATTERY

The DURA-NEB 3000 battery may be recharged using any wall or 12V DC vehicle outlet. Make sure the compressor is turned off. Plug the small round end of the adapter into the socket on the front panel marked "Power Input". Plug the other end of the Charger/Adapter into the wall or vehicle outlet. If using a vehicle outlet, the vehicle must be running in order to recharge the battery. If using a wall outlet, make sure it is not being shared by another power consuming appliance. To ensure the battery receives a complete charge, recharge unit overnight. Recharging may take up to 8 hours, depending on the current charge level of the battery.

D REMOVING THE INTERNAL BATTERY

1. Turn compressor face down on a flat surface.
2. Gently lift up loose end of grey trim to expose battery access door **D-1**.
3. Insert flat screwdriver, coin, spoon handle, or flat object into door notch.

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4. Gently push in and LIFT screwdriver UP to open battery door. Remove door by lifting up **D-2**.
5. Tap compressor against your palm at door opening until battery is dislodged into your hand. DO NOT pry battery out with screwdriver or any other object. If it is difficult to dislodge battery, tap compressor against hard surface until battery is dislodged into palm **D-3**.

▲ WARNING

Do not store medication, nebulizer, or other objects in battery compartment if no battery is installed.

D REPLACING THE INTERNAL BATTERY

1. Turn compressor face up.
2. Insert battery, prongs first, as illustrated **D-4**.
3. Replace battery door.
4. Press grey trim back into place.
5. Charge new battery before first use (see section VII, To Recharge the Internal Battery).

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